

# ALARMED CABINET - BATTERY INSTALLATION GUIDE




Welcome to the simple step by step guide of changing your batteries in your AED Authority alarmed cabinet.

## STEP 1:

Loosen the 2 holding screws as highlighted in figure 1.

Remove these screws to release the battery pack.

Figure 1

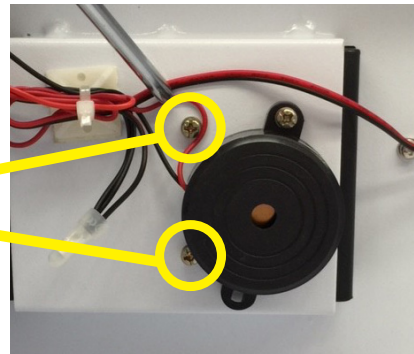


## STEP 2 :

Cut the cable tie with a pair of scissors, the location of the cable tie has been highlighted in figure 2.

By cutting the cable tie you can now maneuver the battery pack freely and remove the battery pack from its shelf.


Figure 2

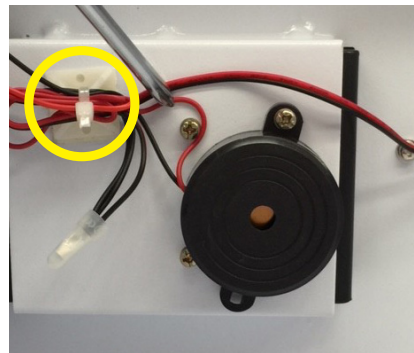

## STEP 3 :

Slide the black battery pad to the right as indicated in figure 3.

Figure 3

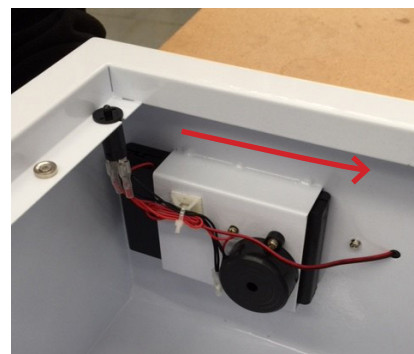


# ALARMED CABINET - BATTERY INSTALLATION GUIDE

## STEP 4 :

As indicated in figure 4 you now have removed the battery pack and are free to remove the black battery cover, to do this slide the cover down as indicated on the front of the pack.

Once you have removed the old batteries, you can proceed with replacing them.


**Figure 4**

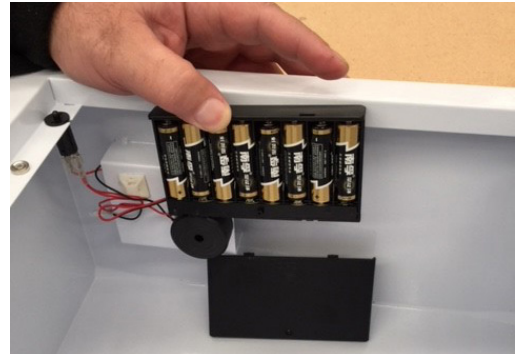

## STEP 5 :

Replace black cover and place back in mounting shelf.


**Figure 5**

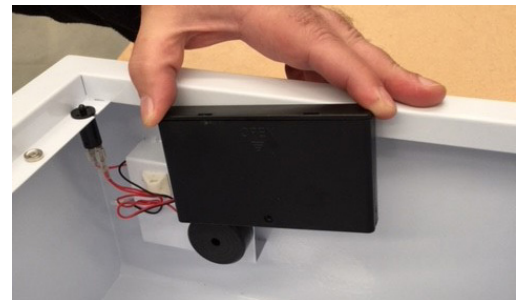

## STEP 6 :

Slide the battery pack back into the shelf towards the left as indicated in figure 6.

**Figure 6**

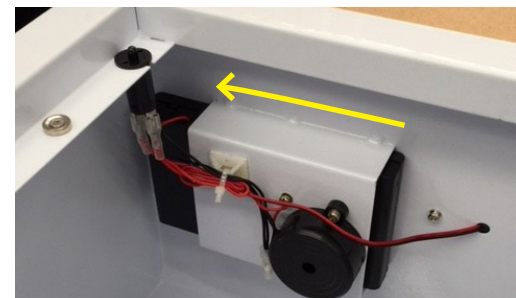

## STEP 7 :

Fasten the 2 holding screws as highlighted in figure 7 and check the alarm to ensure its operating efficiently.

Fasten these screws to ensure the battery pack remains secure.

**Figure 7**

