



Congratulations on the purchase of your AED Authority Alarmed Defibrillator Cabinet, please find below the simple, easy to follow installation instructions.

## STEP 1: UNPACK CABINET FROM PACKAGING.

Remove your new cabinet from its packaging, assess the unit for any damage that may have occurred during delivery, Once you have assessed that the unit is free from any damage you are ready to proceed to installation.

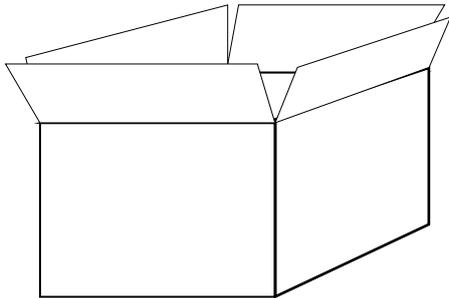


**1a)** Unpack your Cabinet from its packaging.

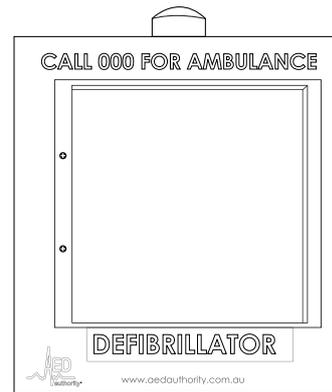


**1b)** Assess all surfaces for damage.

## STEP 2 : REMOVE HANDLE FROM INTERNAL CABINET AND REPLACE ON FRONT OF CABINET.

Open the Defibrillator Cabinet door, once you have opened the door you will find the door handle fixed too the inside of the door, with a Phillips head screw driver remove both screws and the handle. Once this has been done you are free to place the handle on the exterior of the door and replace the screws from the inside of the cabinet.

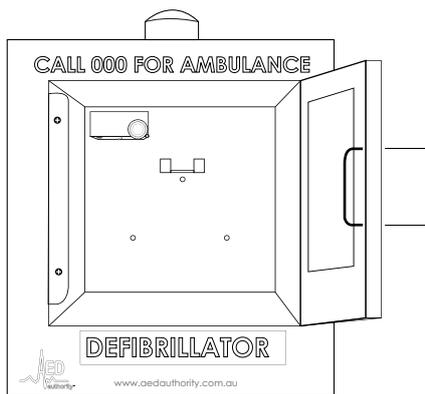

**2a)** Open the door, Notice the handle on the interior side of the cabinet door.

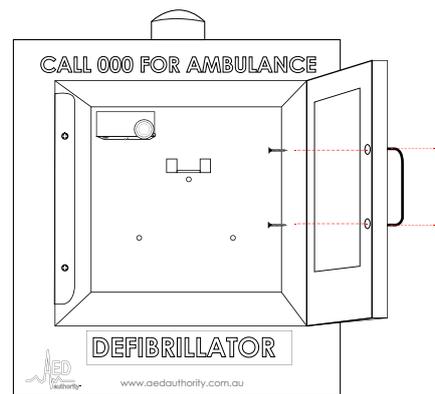

**2b)** Loosen the fixed screws on the outside of the door and remove handle. Once this has been done you can now place handle on the exterior side and fix the cabinet handle correctly.

**AED Authority Australia Pty. Ltd.** ABN: 46 169 556 932 ACN: 169 556 932


(03) 8710 8666 (03) 8710 4700 41 Marconi Drive, Dandenong South, VIC 3175 Australia  
P.O.Box 4147 Dandenong South, Victoria 3164 sales@aedauthority.com.au aedauthority.com.au

Authorised Partner Of Physio Control


## DEFIBRILLATOR ALARMED CABINET - INSTALLATION GUIDE

### STEP 3 : MOUNTING YOU CABINET.

You are now ready to mount your alarmed cabinet, The back panel of the cabinet houses 3 pre-drilled mounting holes. Place the screws through the holes in the back panel and screw into place. Additional equipment may be required depending on the surface the cabinet is being fixed too.

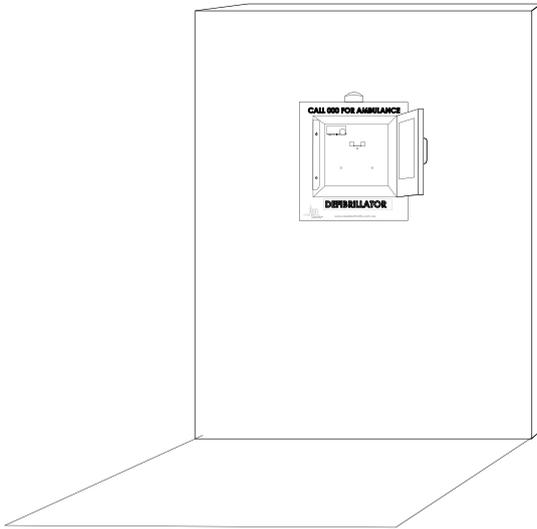

**3a)** Determine where the cabinet is going and mark it on the wall.

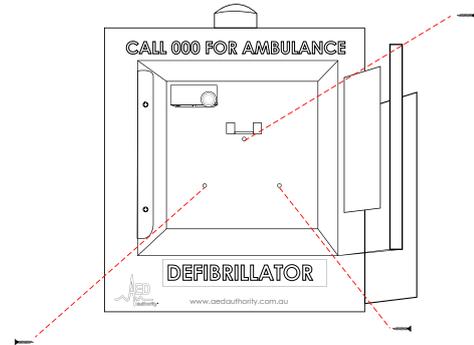


**3b)** Once in the appropriate place, fix the cabinet to the wall by screwing the 3 screws into the mounting holes at the rear of the cabinet.

### STEP 4: TURN ALARM ISOLATOR ON`

Now that you have fixed your handle and mounted your cabinet appropriately it is ready to be alarmed and active. In the top inside of the cabinet you will find the shelf housing the battery pack and isolation switch, Follow the below instructions to make your cabinet active.

**4b)** Reach in and gently slide the switch to on. Once this has been activated the alarm cabinet will be active and sound the alarm tone. To silence the alarm simply close the door.

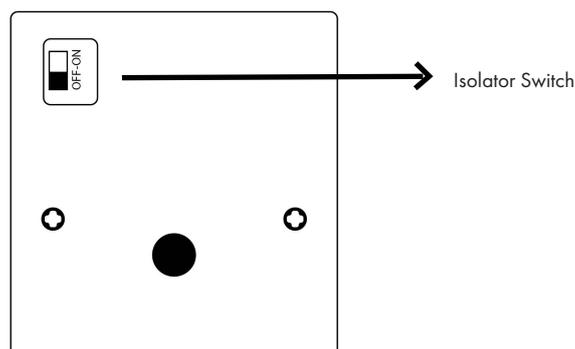

**STEP 5: PLACE AED UNIT IN THE ALARMED CABINET, YOUR UNIT IS NOW SAFELY SECURED & READY FOR USE.**

**AED Authority Australia Pty. Ltd.** ABN: 46 169 556 932 ACN: 169 556 932

(03) 8710 8666 (03) 8710 4700 41 Marconi Drive, Dandenong South, VIC 3175 Australia  
P.O.Box 4147 Dandenong South, Victoria 3164 sales@aedauthority.com.au aedauthority.com.au

Authorised Partner Of Physio Control