

PERFORMING A RESCUE WITH AN AED

(Automated External Defibrillator)



1 Is patient responsive?



2 Call emergency services!
Call 000



3 Get AED.



4 Place AED on ground.
Open lid.



5 Remove clothing.
Expose chest.



6 Attach pads.




7 Do not touch the patient.



8 If required, AED will deliver
shock automatically.



9 Follow AED prompts.
Give compressions.

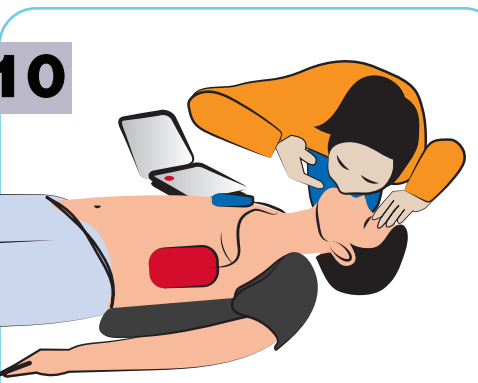


10 If instructed, give breaths.

11 Follow AED
voice prompts until
Emergency Personnel
arrive on the scene.



Simply Saving Lives

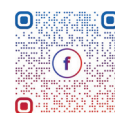

Like us on
Facebook