


# INDOOR ALARMED DEFIBRILLATOR CABINET BATTERY INSTALLATION GUIDE




1. Locate the black battery pack inside the AED Alarmed Cabinet.

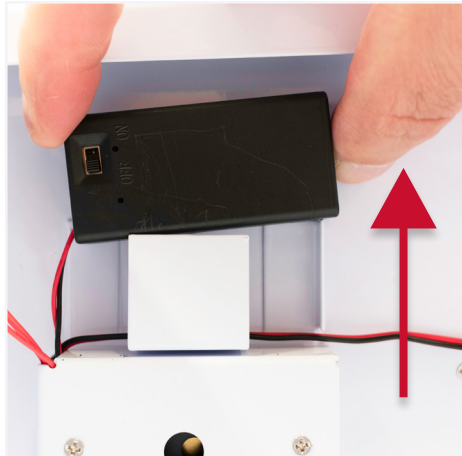


2. Slide the battery pack upwards from the holder.



3. Remove the battery pack cover by sliding it upwards.

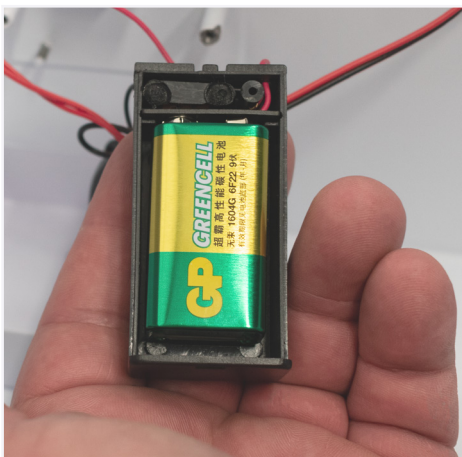

4. Insert the battery and replace the battery pack cover.

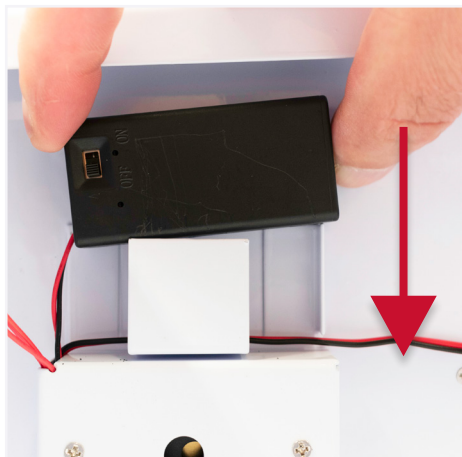

5. Slide down the battery pack into the holder.



6. Please make sure to turn ON the alarm switch.

03 8710 8666

[sales@aedauthority.com.au](mailto:sales@aedauthority.com.au)

41 Marconi Drive, Dandenong South VIC 3175



[aedauthority.com.au](http://aedauthority.com.au)